

StrapStand

5'-6'5"
350 lbs
large



The EasyStand StrapStand has the most innovative lift design of any strap-style stander. Using dual-hinged lifting arms, the StrapStand gently raises a user vertically then gradually pulls them to the standing position reducing knee pressure, unlike other standers that pull a user into the knee pad creating excessive patella pressure.



The straps are designed for maximum comfort and by using a special fabric it is easier to place under a person that is seated. Each strap is sewn to form a pocket that securely supports the user during the sit to stand transition.



Base model



StrapStand with options



Swing-Out Legs
with options



Swing-Out Legs



Independent Knee Pads

The StrapStand's adjustability is well suited for multi-user environments. With popular options like swing-out legs, multi-adjustable foot plates, various sizes and styles of lifting straps, and independent knee pads, a facility can accommodate users of different sizes and diagnoses.




Multi-adjustable Foot Plates



Adjustable Sling Strap

The StrapStand gives people the option to stand without transferring. Often, users are incapable of self-transfers or patient lifting is a concern for staff. In these cases, standing is possible directly from a wheelchair, bed, or other seated surface. In addition, we have adjustable lifting straps that work well for wheelchairs without removable arms and allow access to the tightest wheelchair seating.

StrapStand options

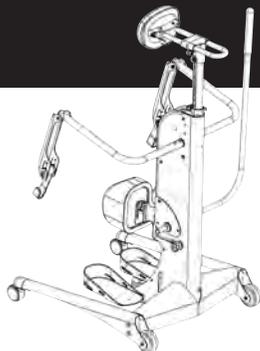

P2100 EasyStand StrapStand

The EasyStand StrapStand base model includes two 3" locking casters, two 4" wheels, foot plates, knee pad, no-tray chest pad, manual hydraulic actuator with handle and black upholstery (lifting strap not included).



80811 Lifting Strap-Small

Small padded lifting strap is 9"Wx25"L (23x64cm).



80719 Lifting Strap-Large

Large padded lifting strap is 12"Wx28"L (30x71cm).



80962 Lifting Strap-XL

XL padded lifting strap is 14"Wx32"L (36x81cm).



80882 Adjustable Lifting Strap-Small

Adjustable lifting strap for fixed arm wheelchairs. Small strap is 9"Wx25"L (23x64cm). Range is 0"-8" (0-20cm).



80881 Adjustable Lifting Strap-Large

Adjustable lifting strap for fixed arm wheelchairs. Large strap is 12"Wx28"L (30x71cm). Range is 0"-8" (0-20cm).



80963 Adjustable Lifting Strap-XL

Adjustable lifting strap for fixed arm wheelchairs. XL strap is 14"Wx32"L (36x81cm). Range is 0"-8" (0-20cm).



82205 Adjustable Sling Strap-Small

Adjustable sling type strap for individuals using fixed arm wheelchairs. Small strap is 9"Wx25"L (23x64cm). Range is 0-8" (0-20cm).



82206 Adjustable Sling Strap-Large

Adjustable sling type strap for individuals using fixed arm wheelchairs. Large strap is 12"Wx28"L (30x71cm). Range is 0-8" (0-20cm).



82207 Adjustable Sling Strap-XL

Adjustable sling type strap for individuals using fixed arm wheelchairs. XL strap is 14"Wx32"L (36x81cm). Range is 0-8" (0-20cm).



P82261 Narrow Clear Tray

Narrow clear acrylic tray fits through narrow doorways. Adjustable in height and depth. Tray size is 18"x19" (46x48cm). Width of the narrow lifting arms is 29" (74cm).



P82262 Narrow Angle Adjustable Tray

Angle adjustable clear acrylic tray fits through narrow doorways. Adjusts from 0-35° to provide forearm support and angle activities closer to the user. Tray size is 18"x19" (46x48cm). Width of the narrow lifting arms is 29" (74cm).



PNG50393 Black Molded Tray

Black molded tray provides anterior support and work surface. Adjustable in height and depth. Tray size is 21"x24" (53x61cm).



PNG50394 Clear Tray

Clear acrylic tray provides anterior support and work surface. Adjustable in height and depth. Tray size is 21"x24" (53x61cm).



PNG50397 Black Molded Angle Adjustable Tray

Adjusts from 0-35° to provide forearm support and angle activities closer to the user. Tray size is 21"x24" (53x61cm).



PNG50398 Clear Angle Adjustable Tray

Adjusts from 0-35° to provide forearm support and angle activities closer to the user. Tray size is 21"x24" (53x61cm).



PNG50042 Hand Grips

Provides additional arm and hand support and alignment. Easily movable for precise positioning. Attaches to clear acrylic tray with suction cups.



P5030 Arm Troughs

Provides additional forearm support and alignment. Easily movable for precise positioning. Attaches to clear acrylic tray with suction cups.



PNG30000 Large Contoured Chest Pad

Attaches to tray or no-tray bracket to provide anterior support with a minimal lateral contour. Replaces standard chest pad. Oval pad is 9.5"x15" (24x38cm).



PNG30295 Padded Tray Cover

A removable padded tray cover with drawstring that attaches to the black molded, clear, angle adjustable, or shadow tray. Not available for the narrow clear tray.



P82090 Multi-Adjustable Foot Plates
 Adjusts independently in height from 1"-5" (2.5-13cm) off floor, plantar/dorsi +/-20°, and +15° toe out adjustment. Recommended for users over 5'10".



P82144 XL Foot Plate
 Accommodates larger feet. Each foot placement area is 7"x14" (18x36cm). Recommended for very large individuals.



PNG30030 Foot Straps
 Velcro® and D-ring adjustment holds feet into alignment with two attachment slots for proper foot positioning. Straps fit foot height of 4"-6.5" (10-17cm).



PNG30031 Secure Foot Straps
 Velcro® and D-ring adjustment holds feet securely in place at ankles and toes. Straps fit foot height of 4"- 6.5" (10-17cm).



P82101 Independent Knee Pads
 Width range from center to center is 8"-12" (20-30cm). Depth range from back of heel is 3.5"-8" (9-20cm). Height range from foot plate to top of pad is 17"-22.5" (43-57cm).



P82189 Independent Roho® Knee Pads
 Roho® Dry Floatation® technology. Width range from center to center is 8"-12" (20-30cm). Depth range from back of heel is 3.5"-8" (9-20cm). Height range from foot plate to top of pad is 17"-22.5" (43-57cm).



PNG50044 Quad Grip Handle Extension
 Attaches to the actuator handle, for users with limited dexterity and function. Adjusts for various hand sizes.



PNG50070 T-Style Handle Extension
 Attaches to the actuator handle for users with limited range of motion. Makes the first few pumps easier and descending more accessible. Adds additional 7" (18 cm) in each direction.



P80829 Upper Body Support Strap
 D-ring adjustment, provides additional upper posterior support while lifting and standing. Pad size is 5.5"x24" (14x61cm). Adjustment range from 33"-70" (84-178cm).



P82062 Grab Handles
 Handles attach to the lifting arms to give the user a place to hold onto during the standing transition.



P82160 Transport Handle
 Attaches to center post of unit to provide the caregiver with an easy way to move the unit.



P82188 Swing-Out Legs
 Swing in/out the legs of the stander for wider wheelchair access. For people with wheelchair width greater than 22" (56cm) and up to 38" (97cm). Includes four locking casters.