

## wide range of accessories...

Meywalk® mk3 is as a standard fitted with handlebar, trunk support in one of four sizes, rear stop, a double parking brake and springs to match the body weight of each individual user.

Springs are available in four sizes  
 white: min. 20 kg  
 yellow: min. 45 kg  
 blue: min. 65 kg  
 red: min. 80 kg



A range of accessories are available for Meywalk® mk3. The accessories ensure that the gait trainer will fit the exact needs of the user.

- Swivel locks
- Anti tip support
- Drag brakes
- Non-reverse brakes
- Tray (is fastened to the handlebar)
- Leg guides
- Leg divider

## increased quality of life

Meywalk® mk3 is a gait trainer for people with various disabilities affecting their mobility. The users often sit in a wheelchair and need a large degree of help and support to stand up and walk. Meywalk® mk3 will increase the quality of life for the user remarkably by offering the possibility of being able to actively participate in everyday activities as well as being able to make choices based on own preferences.



## precisely fitted

The seat and trunk support of the gait trainer supports the user and helps to keep the body erect. Both seat and trunk support can be continuously adjusted in height so the gait trainer is fitted precisely to the user and 'grows' with her or him.



## toolfree adjustment


Meywalk® mk3 is available in two sizes and is adjusted to fit the needs and size of each individual user. All settings are done easily without the use of tools, which is a great advantage for therapists and other staff during active and busy days.



## natural gait pattern

Meywalk® mk3 stands out in the market of assistive technology by being spring suspended. The springs stimulate the muscles of the user and the movements that are a part of a natural gait are imitated. This way the gait pattern is developed. In practice the upper part of the frame with seat and trunk support follow the movements of the user. Thereby Meywalk® mk3 provides optimal support in all phases of gait.



## Unique system for getting in and out

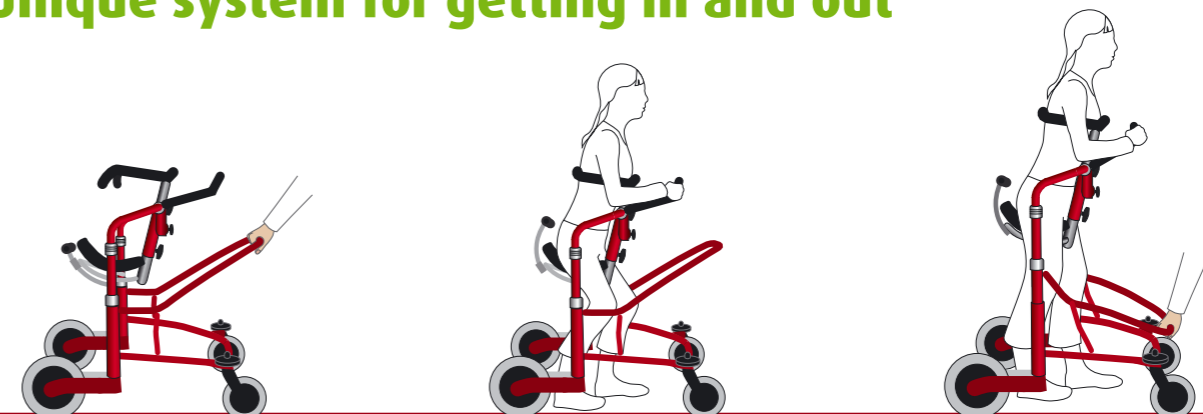



# meywalk® mk3

If you would like to try Meywalk® mk3, you just need to provide us with 3 pieces of information:

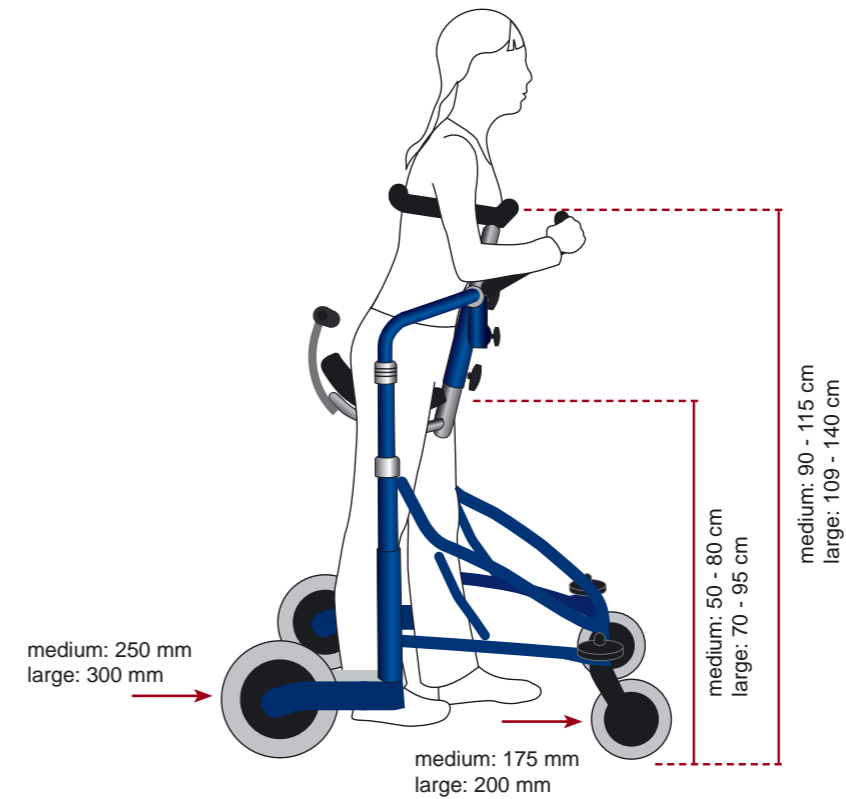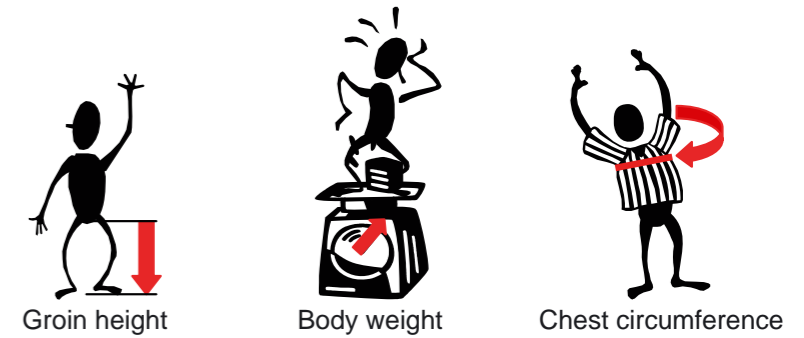

The trunk support for Meywalk® mk3 is available in five sizes: 70 cm, 90 cm, 105 cm, 120 cm and 130 cm circumference.

## FACTS om meywalk® mk3

	medium	large
Length:	85 cm	101 cm
Width	62 cm	69 cm
Maximum load:	100 kg	100 kg
Weight:	30,5 kg	33,5 kg
Order number:	010184	010185

Distributed by:



## Dejay Medical & Scientific Pty Ltd

1 Prince William Drive, Seven Hills, NSW 2147

Telephone: NSW (02) 9838 8869 | VIC (03) 9558 9669 | QLD (07) 3863 2589

[www.dejay.com.au](http://www.dejay.com.au)

meywalk® mk3 is made from steel tubes. It has swivel wheels in the front and is available in two colours, metallic red or metallic blue. (We reserve the right to make changes to design and/or structure)

